



Installation Check List
CardiAssist Model 4000/4500

Account Name: \_\_\_\_\_

Phone#: \_\_\_\_\_ Fax#: \_\_\_\_\_

Contact (s): Manager: \_\_\_\_\_ ECP Tech: \_\_\_\_\_

Machine Serial Number: \_\_\_\_\_

Table with 12 rows of checkboxes and text instructions for installation steps, including: Install quick disconnects on bladders, Plug machine into 20Amp dedicated line, Connect ECG cable to ECG outlet, Connect PPG/ O2 Sat to USB Port on Monitor, Turn on the machine, Put Electrodes on (self, volunteer or simulator) and connect ECG leads -verify ECG input is displayed, Clip PPG on finger- Verify PPG wave form is displayed, Connect hoses to machine air outlets. Attach to cuffs/bladders, Start Treatment Cycle, Voltage Under Load: \_\_\_\_\_, Stop Machine, Select print cycle-Verify printer operates, Exit software and shut down.

Comments \_\_\_\_\_

Installer Signature \_\_\_\_\_

Date

Installer-Please return this form to Quality Assurance Documentation upon completion.